

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
- LF00856U** Female Catheterization
- LF00901U** Prostate Examination
- LF00906U** Ostomy Care
- LF00929U** Surgical Bandaging
- LF00957U** Enema Administration
- LF00958U** Pediatric Injectable Arm
- LF00961U** Intramuscular Injection
- LF00984U** Breast Examination
- LF00995U** Arterial Puncture Arm
- LF00997U** Adult Injectable Arm (Black)
- LF00999U** Pediatric Injectable Head
- LF01008U** Intradermal Injection Arm
- LF01012U** Heart Catheterization (TPN)
- LF01019U** Ear Examination
- LF01020U** Supplementary Ear Set
- LF01025U** Male Cath-Ed I
- LF01026U** Female Cath-Ed II
- LF01027U** Peritoneal Dialysis
- LF01028U** Suture Practice Arm
- LF01036U** Spinal Injection
- LF01053U** Cross-Sectional Anatomy, Torso, Head
- LF01054U** Cross-Sectional Anatomy, Head
- LF01062U** Pelvic, Normal & Abnormal
- LF01063U** Stump Bandaging, Upper
- LF01064U** Stump Bandaging, Lower
- LF01069U** Cervical Effacement
- LF01070U** Birthing Station
- LF01082U** Cricothyrotomy
- LF01083U** Tracheostomy Care
- LF01084U** Sigmoidoscopic Examination
- LF01087U** Central Venous Cannulation
- LF01095U** Blood Pressure Arm
- LF01108U** Intraosseous Infusion Simulator
- LF01142U** Auscultation Trainer
- LF01162U** Venatech IV Trainer
- LF03000U** CPARLENE® Series
- LF03601U** Adult Airway Management Trainer
- LF03602U** Adult Airway Management on Manikin
- LF03603U** Adult Airway Management Head Only
- LF03609U** Child Airway Management Trainer
- LF03610U** Child Airway Management Trainer Head Only
- LF03611U** Child Defibrillation Chest Skin
- LF03612U** Child IV Arm
- LF03613U** Child Blood Pressure Arm
- LF03614U** Child Intraosseous Infusion/ Femoral Access Leg Only
- LF03615U** Complete Child *CRiSis*™ Update Kit
- LF03616U** Child *CRiSis*™ Manikin
- LF03617U** Deluxe Child *CRiSis*™ Manikin with Arrhythmia Tutor
- LF03620U** PALS Update Kit
- LF03621U** Infant Airway Management Trainer Head Only
- LF03622U** Intraosseous Infusion Right Leg
- LF03623U** Infant Airway Management Trainer
- LF03626U** Child Femoral Access Injection Pad Replacement
- LF03632U** Child Intraosseous Infusion/ Femoral Access Leg on a Stand
- LF03633U** Child Airway Management Trainer with Torso
- LF03693U** **Basic Buddy** CPR Manikin
- LF03699U** "Airway Larry" Airway Management Trainer
- LF03720U** **Baby Buddy** Infant CPR Manikin
- LF03953U** *CRiSis*™ Manikin
- LF03955U** Deluxe *CRiSis*™ Manikin
- LF04001U** *GERi*™ Nursing Manikin
- LF04020U** *KERi*™ Nursing Manikin
- LF04021U** *KERi*™ Basic Manikin
- LF04022U** *KERi*™ Advanced Manikin
- LF04030U** *GERi*™ Advanced Manikin
- LF04040U** *GERi*™ Basic Manikin

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com

COPYRIGHT © NASCO 1999 PRINTED IN U.S.A. NP 25-01

Life/form®

PATIENT EDUCATION INFANT TRACHEOSTOMY CARE SIMULATOR LF01167U INSTRUCTION MANUAL



Life/form®

About the Simulator...



The **Life/form**® Patient Education Infant Tracheostomy Care Simulator is ideal for teaching caregivers the skills they will need to perform at home. Students can practice suctioning techniques, proper cuff inflation, dressing changes and other techniques. Cleansing and maintenance of the external tracheal site can be performed just as on an actual patient. The infant manikin can be used to teach caregivers basic tracheostomy care skills.

List of Components

1. Infant Tracheostomy Care Simulator
2. Pediatric Tracheostomy Tube
3. Carrying/Storage Case
4. REN Cleaner
5. NASCO **Life/form**® Lubricant

General Information about the Simulator

Procedures that may be performed on the Infant Simulator:

- Dressing changes
- Stoma cleansing
- Changing of tracheostomy tube
- Cuff inflation
- Changing tracheostomy ties

The simulator may be used in basic methods of tracheostomy care. Individual methods and procedures may differ; therefore this booklet does not propose to outline a particular procedure to be followed. Rather, it is recommended that

a procedural manual and instructor's directions be followed. This will eliminate confusion and best serve the individual instructor and curriculum.

Care of the Simulator

Normal surface soil can be removed with a mild soap and luke-warm water. Use REN cleaner (W09919UA) to remove stubborn stains. Simply apply the REN cleaner to the soiled area and wipe with a clean, soft paper towel.

Cautions

1. Do not cleanse the stoma area with Betadine or any other similar, colored disinfectant. NEVER place the trainer on any kind of printed paper or plastic. These materials, as well as ballpoint pens, will transfer indelible stains. Do not use any cosmetics on the simulator.

2. Exercise the same care with this simulator as would be used with an actual patient. Excessive pressure and abuse will damage and shorten the unit's life.

Supplies/Replacement Parts for the Patient Education Infant Tracheostomy Care Simulator

W09919UA REN Cleaner
LF00985UA Lubricant Kit
LF1171U Pediatric Trachea Tube