

"FAT OLD FRED" MANIKIN LF03750U INSTRUCTION MANUAL



Life/form Products by NASCO

Life/form® Fat Old Fred CPR Manikin



The first and only CPR manikin to offer the realistic size and appearance of an older, heavier cardiac victim — the kind rescue personnel are most likely to encounter in the field!

Accurate anatomical landmarks and ease of use will make Fat Old Fred a valuable addition to your training program. Sanitary features include individual face inserts, disposable airway system, and a non-rebreathing valve. For added convenience and economy, all internal parts (including the airway system) are fully compatible with the Simulaids™ Brad CPR Manikin.

Removing the Used Airway:

- Remove the face insert and connector from the face cavity.
- 2. With the manikin face up, pull the overlay skin from the shoulders and fold down. Lift off the sternum piece and foam pad to expose the lung.
- 3. Tilt the head back and pull out the valve from underneath.
- 4. Remove and discard the lung and valve.

Installing the New Airway:


- Press the red exhaust manifold cap onto the open port of the airway valve, next to the port that is attached to the lung.
- 2. Tilt the head back and insert the valve stem through the hole in the face from underneath.

NOTE: In order for the simulator to work properly, the airway connection must be toward the bottom of the manikin, with the exhaust manifold above it.

- Position the connector on top of the face and press onto the valve stem with a twisting motion. Recheck the airway position.
- Arrange the lung over the chest so that it lies flat.
- 5. Replace the foam pad, sternum, and the overlay skin.
- 6. Snap in a clean face insert.

Cleaning and Disinfecting:

- All vinyl parts (overlay skin, body, head, face inserts) may be washed with water and a mild detergent. Do not attempt to wash the foam pad.
- 2. NASCO recommends disposing of the face inserts after each training session. They may be reused after thorough cleaning if desired:
 - wash carefully in mild soapy water
 - rinse thoroughly
 - disinfect by immersing completely in a solution of 1 teaspoon chlorine bleach in 1 gallon of water never use undiluted bleach
 - rinse thoroughly

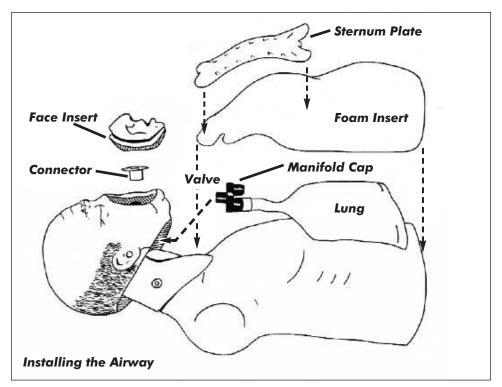

Replacement Components:

LF03750U

Fat Old Fred with 3 mouth/nose pieces, 3 disposable lung/airway systems, and carrying bag.

LF03751U

Lung/Airway Systems, pkg. of 24.

LF03752U

Mouth/Nose Pieces, pkg. of 10.

Other Available $\textit{Life/form}_{\circ}$ Simulators

Other A	lvailable <i>Life/form</i>	Simulators	
LF00698U	Adult Injectable Arm (White)	LF03610U	Child Airway Management
	Female Catheterization '		Trainer Head Only
LF00901U	Prostate Examination	LF03611U	Child Defibrillation Chest
LF00906U	Ostomy Care		Skin
LF00929U	Surgical Bandaging	LF03612U	
	Enema Administration	LF03613U	
LF00958U	Pediatric Injectable Arm	LF03614U	Child Intraosseous Infusion/
	Intramuscular Injection		Femoral Access Leg Only
	Breast Examination	LF03615U	Complete Child CRISIS™
LF00995U	Arterial Puncture Arm		Update Kit
LF00997U	Adult Injectable Arm (Black)	LF03616U	Child CRISIS™ Manikin
LF00999U	Pediatric Injectable Head	LF03617U	Deluxe Child CRISIS™
LF01008U	Intradermal Injection Arm		Manikin with Arrhythmia Tutor
	Heart Catheterization (TPN)	LF03620U	PALS Update Kit
	Ear Examination	LF03621U	Infant Airway Management
LF01020U	Supplementary Ear Set		Trainer Head Only
	Male Cath-Ed I	LF03622U	Intraosseous Infusion Right
	Female Cath-Ed II		Leg
LF01027U	Peritoneal Dialysis	LF03623U	Infant Airway Management
LF01028U	Suture Practice Arm	1500/0/11	Trainer
LF01036U	Spinal Injection	LF03626U	Child Femoral Access
LF01053U	Cross-Sectional Anatomy,	1502/0711	Injection Pad Replacement
150105411	Torso, Head Cross-Sectional Anatomy,	LF03627U	Child CRISIS™ Replacement
LF010540	Head	LF03628U	Lung Set Child CRISIS™ Replacement
150106211	Pelvic, Normal & Abnormal	LFU3020U	Stomach Replacement
LF01062U	Stump Bandaging, Upper	LF03629U	Child IV Arm Replacement
LF01064U	Stump Bandaging, Lower	EI 000270	Skin and Veins
LF01064U	Brachial Plexus	LF03630U	Child Arrhythmia Tutor
	Cervical Effacement	LF03631U	Child Airway Management
LF01070U	Birthing Station	1 1 000010	Trainer Replacement
LF01082U	Cricothyrotomy		Headskin
LF01083U	Tracheostomy Care	LF03632U	Child Intraosseous Infusion/
LF01084U	Sigmoidoscopic		Femoral Access Leg on a
	Examination		Stand
LF01087U	Central Venous Cannulation	LF03633U	Child Airway Management
LF01094U	Cross Sectional		Trainer with Torso
	Anatomy — Laminated	LF03650U	CRISIS™ Manikin
LF01095U	Blood Pressure Arm	LF03651U	ALS Update Kit
LF01108U	Intraosseous Infusion	LF03693U	Basic Buddy™ Manikin
	Simulator	LF03694U	Basic Buddy™ 5-Pack
	CPARLENE® Series	LF03695U	Basic Buddy™ 10-Pack
LF03601U	Adult Airway Management	LF04001U	GERi™ Nursing Manikin
	Trainer	LF04020U	KERi™ Nursing Manikin
LF03602U	Adult Airway Management	LF04021U	
	on Manikin	LF04022U	
LF03603U	Adult Airway Management	LF04030U	
	Head Only	LF04040U	GERi™ Basic Manikin
IEUSYUOII	Child Ainway Managament		

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901 Fort Atkinson, Wisconsin 53538-0901 1-800-558-9595

LF03609U Child Airway Management Trainer

www.eNASCO.com • E-Mail lifeform@eNASCO.com